# MAS® TLIF

# Maximum Access Surgery Transforaminal Lumbar Interbody Fusion


This booklet provides general information on MAS TLIF. It is not meant to replace any personal conversations that you might wish to have with your physician or other member of your healthcare team. Not all the information here will apply to your individual treatment or its outcome.




## About the spine

The human spine is made up of 24 bones or vertebrae in the cervical (neck) spine, the thoracic (chest) spine, and the lumbar (lower back) spine, plus the sacral bones.

Vertebrae are connected by several joints, which allow you to bend, twist, and carry loads. The main joint between two vertebrae is called an intervertebral disc. The disc is made of two parts, a tough and fibrous outer layer (annulus fibrosis) and a soft, gelatinous center (nucleus pulposus). These two parts work in conjunction to allow the spine to move, and also provide shock absorption.

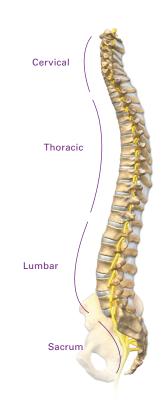



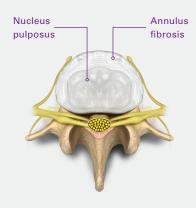


# About the spinal cord and cauda equina


Each vertebra has an opening (vertebral foramen) through which a tubular nervous structure travels. Beginning at the base of the brain to the upper lumbar spine, this structure is called the spinal cord.

Below the spinal cord, in the lumbar spine, the nerves that exit the spinal cord continue to travel through the vertebral foramen as a bundle known as the cauda equina.

At each level of the spine, spinal nerves exit the bony spine then extend throughout the body.

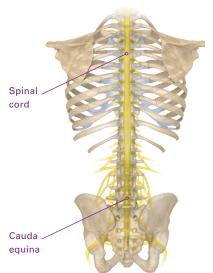



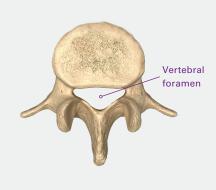

## What can cause pain?

There are several primary causes of spine problems. The majority of the symptoms are caused by either instability or by disc, bone or ligaments pressing onto the nerve roots, spinal cord, and/or cauda equina.

### Degenerative Disc Disease (DDD)

During the natural aging process, the discs between each vertebral body can lose their flexibility, height, and elasticity which can cause a tear in the tough outer layer of the disc, causing the disc to herniate, bulge, or leak the gelatinous core. The bulges or leakages can end up compressing the nerve roots, spinal cord, and/or cauda equina causing symptoms including, but not limited to lower back and/or leg pain.

### Degenerative Spondylolisthesis

Degenerative spondylolisthesis is a condition where one vertebra has slipped forward over another one below it. This instability typically occurs as a result of degenerative changes but may also be caused by stress fractures, or congenital abnormalities (birth defects), and in rare cases from a tumor or trauma.

## **Degenerative Scoliosis**

Adult degenerative scoliosis is a condition where a right-left or lateral curve develops in a previously straight spine. This curvature occurs as a result of deterioration of the disc and joints in the back of the spine. As the joints degenerate they create a misalignment in the back, resulting in a bend or curvature, causing symptoms including lower back and/or leg pain.

## What are the treatment options?

Many symptoms can be treated without surgery including rest, heat, ice, medication, injections, and physical therapy. It is important to speak with a physician about the best option.

If symptoms do not improve with conservative treatment, physicians may recommend spinal surgery. Surgery is reserved for those who do not gain relief from non-operative forms of treatment, patients whose symptoms are increasing or worsening, and/or patients that present with a spinal condition which indicates the need for surgery.

## What is a MAS TLIF procedure?

Lumbar interbody fusion is a surgical technique that attempts to eliminate instability in the back. A MAS TLIF achieves this by using a less invasive approach to fuse one or more vertebrae together to reduce their motion.

Rather than starting from the middle of the back and spreading the muscles to the sides as is done in traditional back surgery, the MAS TLIF approach starts off to one side of the spine and splits (rather than cuts) the back muscles in one direction. This allows the surgeon to make a smaller incision with less muscle injury, which may result in less postoperative pain and a quicker recovery.

# Can a MAS TLIF be right for me?

Your physician might determine a MAS TLIF procedure is a good option for you if you require a spinal fusion at any lumbar level between L1 and S1, and you would benefit from a less disruptive approach than traditional open techniques.

Conversely, your physician may determine that a MAS TLIF procedure is not a good option for you. It is important to discuss this with your physician in order to determine the best course of treatment for you.

## What to expect

## Before surgery

Your physician will review your condition and explain treatment options, including medications, physical therapy, and other surgeries. Should you have any questions regarding the procedure, do not hesitate to ask your surgeon. Your physician will provide thorough preoperative instructions.

### **During surgery**

After you are sedated, positioned on your stomach, and surrounded by the appropriate surgical draping, an X-ray image is taken of your spine to identify the location of the operative disc space.

#### Step 1: Approach

Typically, your surgeon will make a small incision on one side of your back. The size of the incision can vary based on the number of levels and/or complexity of your case.

#### Step 2: Disc removal

The diseased or damaged disc is removed to reduce pressure from the cord and/or symptomatic nerve root.

#### Step 3: Implant


An appropriate implant, chosen by your surgeon, will be placed into the disc space to restore the proper disc height and provide mechanical support while bone grows between the vertebral bodies during the fusion (bone healing) process. That segment of your spine will eventually stabilize once fusion occurs.

#### Step 4: Fixation

Screws will be placed into the vertebrae and connected by rods to provide stabilization for the spine while fusion occurs. Once the screws have been placed on the MAS TLIF side, screws are also inserted on the opposite side of the vertebrae through one or two small skin incisions.

## What implants are used?

Below are some examples of the implants that may be used during your MAS TLIF procedure:

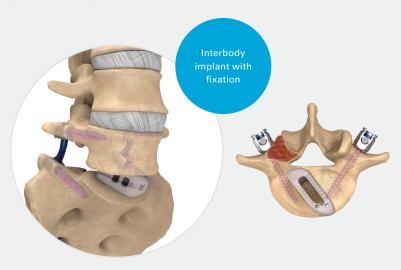

## After surgery

After surgery you will wake up in the recovery room, where your vital signs will be monitored and your immediate postoperative condition will be carefully observed. Once the medical staff feels that you are doing well, you will be returned to your room in the hospital.

Your physician will determine the best postoperative course for you. The day after your surgery, your physician may instruct you to use a brace for a period of time to assist with the spinal fusion process. Supervised by trained medical professionals, your physician may ask you to carefully sit, stand, or walk. Your physician will also discuss with you any medications to take home, as well as a prescribed program of activities. Your physician will provide instructions on wound care, exercises, and limitations to postoperative activity.

# What are the potential benefits of a MAS TLIF procedure?

When compared to traditional open techniques, MAS TLIF has:

- Reduced blood loss<sup>1-3</sup>
- Shorter hospital stay<sup>1-3</sup>
- Smaller incision
- Goldstein CL, Phillips FM, Rampersaud YR. Comparative effectiveness and economic evaluations of open versus minimally invasive posterior or transforaminal lumbar interbody fusion. Spine 2016;41(8S)S74-89.
- Sembrano, JN, Tohmeh A, Isaacs R, et al. Two-year Comparative Outcomes of MIS Lateral and MIS Transforaminal Interbody Fusion in the Treatment of Degenerative Spondylolisthesis. Spine 2016;41(8S):123-32.
- Isaacs RE, Sembrano JN, Tomeh AG, et al. Two-Year Comparative Outcomes of MIS Lateral and MIS Transforaminal Interbody Fusion in the Treatment of Degenerative Spondylolisthesis. Spine 2016; 41(8S):133-44.

# What are the potential risks of a MAS TLIF procedure?

Keep in mind that all surgery presents risks and complications that are important to discuss with your surgeon prior to your surgery. Listening to your physician's guidance, both before and after surgery, will help your recovery.

Potential complications following MAS TLIF surgery include: problems with anesthesia, infection, nerve damage, problems with the graft or hardware, and ongoing pain. This is not intended to be a complete list of the possible complications. Please contact your physician to discuss all potential risks.

## Frequently asked questions

#### Can I shower after surgery?

Depending on your surgical incision, you may have showering restrictions. Ask your physician for appropriate instructions.

#### Will I have a scar?

Your physician will discuss the incisions that will be made during a MAS TLIF surgery.

#### When can I drive?

For a period of time after your surgery, you may be cautioned about activities such as driving. Your physician will tell you when you may drive again.

#### Can I travel?

The implants used in the MAS TLIF procedure may activate a metal detector. Because of increased airport security measures, please call your local airport authority before traveling to get information that might help you pass through security more quickly and easily. Ask your physician to provide a patient identification card.

Notes
Resources
For more information about MAS TLIF, please visit: nuvasive.com
If you would like to learn more about patient support and education for chronic back, leg, and neck pain sufferers and their loved ones, please visit:
thebetterwayback.org

If you have any questions about MAS TLIF or spine surgery, please call or visit your physician, who is the only one qualified to diagnose and treat your spinal condition. This patient information brochure is not a replacement for professional medical advice.



## About The Better Way Back®

The Better Way Back is a nationwide patient support program created by NuVasive\*, a leader in developing minimally invasive, procedurally-integrated spine solutions. The Better Way Back is a free community built on the power of empathy, and is dedicated to providing hope, support, and information to individuals suffering from chronic back, leg, or neck pain.

Through its Patient Ambassador Program, The Better Way Back pairs patients considering spine surgery with patients who have previously undergone a spine procedure. Ambassadors volunteer their time to discuss their experiences in order to provide additional, first-hand perspectives.

To learn more about The Better Way Back, please



call **1-800-745-7099** 



visit thebetterwayback.org



text "TBWB" to 858-360-8292

# **MAS TLIF**

#### NuVasive, Inc.

7475 Lusk Blvd., San Diego, CA 92121 +1 800.475.9131

©2019. NuVasive, Inc. All rights reserved. The Better Way Back is a registered trademark of NuVasive, Inc. 9501028 C

